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Yuyao Tongji Medical Instrument Co., Ltd. was established in October 2011. It is a professional R & D, production and sales of orthopedic orthopedic rehabilitation equipment company, located in Yuyao City, Zhejiang Province, from Shanghai Pudong International Airport about 3 hours by car from the Xiaoshan International Airport , Ningbo Lishe International Airport about 1 hour drive, the company's main products are Tongji adjustable neck care, waist circumference, all kinds of brace, apply to the human body's neck, trunk and limbs of the muscles, ligaments and joints of various injuries, The product is made of non-toxic non-toxic new polymer material, with anti-bacterial, anti-mildew, antistatic properties, its insulation, protection, support three basic characteristics and meet the human anatomy, physiological and motor characteristics, set treatment, rehabilitation , Sports protection in one, selling the world more than 10 countries and regions. The company also undertake all kinds of brazing manufacturers, the company's OEM orders and dedication to customer service!

Care of your every moment !

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The patent certificate

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享受生活 爱护自己
Enjoy life take good care of yourself

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Cervical collar TJ-JT

Product Features

- ◆ Superior immobilization
- ◆ Anti-mildew, anti-bacterial, anti-static
- ◆ Compatible with all imaging modalities

Indications

- ◆ Prevention for cervical trauma patients
- ◆ Immobilization for pre and post c-spine surgery
- ◆ Cervical degenerative disease
- ◆ Cervical stenosis
- ◆ Cervical spondylolisthesis

NOTE

- ◆ The product is advised for single patient, do notvre-use for more than one patient
- ◆ Size choices should be based on the patient' s neck length, do not based on tall

Ordering Information

Specification	Neck height (cm)
L	15
M	14
S	13
XS	12
Note:Colors and special specifications can be customized.	

Conventional

Cervical collar TJ-JT

Product Features

- ◆ Superior immobilization
- ◆ Anti-mildew, anti-bacterial, anti-static
- ◆ Compatible with all imaging modalities

Indications

- ◆ Prevention for cervical trauma patients
- ◆ Immobilization for pre and post c-spine surgery
- ◆ Cervical degenerative disease
- ◆ Cervical stenosis
- ◆ Cervical spondylolisthesis

NOTE

- ◆ The product is advised for single patient, do notvre-use for more than one patient
- ◆ Size choices should be based on the patient' s neck length, do not based on tall

Ordering Information

Specification	Neck height (cm)
L	15
M	14
S	13
XS	12
Note:Colors and special specifications can be customized.	

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Lumbar holder TJ-YZ-01

Product Features

- ◆ This product is breathable mesh fabric, wearing comfortable, durable; Can close or clothes to wear
- ◆ Large TAB is easy to be found, when using fingers clasp and pull it hard
- ◆ Suitable for waist circumference 50–120cm

Indications

- ◆ Various acute and chronic waist injury
- ◆ Lumbar disc disease, lumbar spinal postoperative rehabilitation
- ◆ Lumbar bone hyperplasia, osteoporosis caused by low back pain

NOTE

- ◆ High fever, bleeding tendency, wearing parts skin lesions are prohibited
- ◆ Acute soft tissue injury after 24 hours of use
- ◆ Allergy to this product with caution, may cause the skin redness, burning phenomenon, take out the product and phenomenon disappear naturally
- ◆ It is forbidden to the water soak, usable water gently cleansing. Do not use detergent, bleach washing

Ordering Information

Specification	size	waistline(cm)
XXL	130×22	100-120
XL	120×22	94-107
L	110×22	80-93
M	100×22	67-84
S	90×22	50-70
Note:Colors and special specifications can be customized.		

Lumbar holder TJ-YZ-02

Product Features

- ◆ This product is breathable mesh fabric, wearing comfortable, durable; Can be worn on clothes
- ◆ Large TAB is easy to be found, when using fingers clasp and pull it hard
- ◆ Suitable for waist circumference 50–120cm

Indications

- ◆ Various acute and chronic waist injury
- ◆ Lumbar disc disease, lumbar spinal postoperative rehabilitation
- ◆ Lumbar bone hyperplasia, osteoporosis caused by low back pain

NOTE

- ◆ High fever, bleeding tendency, wearing parts skin lesions are prohibited
- ◆ Acute soft tissue injury after 24 hours of use
- ◆ Allergy to this product with caution, may cause the skin redness, burning phenomenon, take out the product and phenomenon disappear naturally
- ◆ It is forbidden to the water soak, usable water gently cleansing. Do not use detergent, bleach washing

Ordering Information

Specification	size	waistline(cm)
XXL	130×22	100-120
XL	120×22	94-107
L	110×22	80-93
M	100×22	67-84
S	90×22	50-70
Note:Colors and special specifications can be customized.		

Lumbar holder TJ-YZ-03

Product Features

- ◆ Double-layer fabric with no elastic force
- ◆ Not easy to stick stains, easy to clean
- ◆ Suitable for waist circumference 53–122cm

Indications

- ◆ Strong support for the waist, improve spinal stability and reduce stress
- ◆ Has a good effect on the contraction of the abdomen for postpartum recovery

NOTE

- ◆ High fever, bleeding tendency, wearing parts skin lesions are prohibited
- ◆ Acute soft tissue injury after 24 hours of use
- ◆ Allergy to this product with caution, may cause the skin redness, burning phenomenon, take out the product and phenomenon disappear naturally
- ◆ It is forbidden to the water soak, usable water gently cleansing. Do not use detergent, bleach washing

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Medical bracer TJ-JZ-01

Product Features

- ◆ Superior immobilization
- ◆ Short cylinder type or long cylinder type
- ◆ Unique foot shock-absorbing function, reduce the shock of walking effectively

Indications--long cylinder type

- ◆ II、III grade of ankle sprain
- ◆ The soft tissue trauma
- ◆ Stability fracture (calcaneus, talus, phalanges, phalanges, etc.)
- ◆ Bone fracture
- ◆ After operation(within a fixed period)
- ◆ Trauma and rehabilitation period

Ordering Information

Specification	Full length(mm)	European code
L	245-280	39.5-44
M	220-245	36-39.5
S	< 220	< 36
Note:Colors and special specifications can be customized.		

Medical bracer TJ-JZ-02

Product Features

- ◆ Superior immobilization
- ◆ Short cylinder type or long cylinder type
- ◆ Unique foot shock-absorbing function, reduce the shock of walking effectively

Indications--short cylinder type

- ◆ I、II grade of ankle sprain
- ◆ Stable fracture and soft tissue injury
- ◆ Foot trauma
- ◆ Rehabilitation period(such as:cyst excision,hallux valgus osteotomy operation,etc.)

Ordering Information

Specification	Full length(mm)	European code
L	245-280	39.5-44
M	220-245	36-39.5
S	< 220	< 36
Note:Colors and special specifications can be customized.		

Medical bracer TJ-JZ-03

Product Features

- ◆ Two adjustable rear side airbag devices
- ◆ Foot bottom decompression technology
- ◆ Light curved soles, experimentally proven to help simulate natural gait

Indications--short cylinder type

- ◆ Metatarsal fracture
- ◆ Foot front and middle foot injuries
- ◆ Ankle and foot soft tissue damage

Ordering Information

Specification	Full length(mm)	European code
L	275-305	43-47
M	240-275	39-43
S	215-240	35-39
Note:Colors and special specifications can be customized.		

Foot fixator TJ-ZB

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此款专为糖尿病足，足部骨折，
拇外翻手术后特制矫形鞋
This special orthopaedic shoe is
specially designed for diabetic foot,
foot fracture and hallux valgus
surgery.

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Order information:

Model	TJ-ZB				
Specifications (code)	35-36	37-38	39-40	41-42	43-44
Foot length (mm)	225-230	235-240	245-250	255-260	265-270
Note: Colors and special specifications can be customized.					

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